



LikoStretch 1900™



Safe and Quick lifting in a horizontal position

Transferring patients in a horizontal position, for instance in connection with surgery, radiology or intensive care, is often a challenge. It should be quick, simple and ergonomic for caregivers and comfortable and safe for the patient.

LikoStretch 1900 meets all these requirements. The stretcher is connected to a lift and the patient can therefore be transferred in a horizontal position between, e.g., bed and gurney without strenuous, manual lifting for the caregivers. Also, patient care, such as hygiene and changing sheets, is facilitated if the patient can be lifted with a stretcher instead of manually. Furthermore, the open design allows other equipment, e.g., tubes for respiratory aid, to be connected during the entire transfer operation.

LikoStretch 1900 weighs only 19 kg (42 lbs.) and can be folded for convenient storage.

Fits Several Lifts

LikoStretch 1900 can be used together with the overhead lifts Likorall and Multirall, and the mobile lifts Viking L, Viking XL and Golvo.

Accessories

Quick-Release Hook

If the lift and LikoStretch 1900 are equipped with a Quick-Release Hook, the stretcher can be assembled easily when need arises, without the use of tools. LikoStretch 1900 can also be combined with LikoScale to weigh patients in connection with lifting.

Stretch Leveller

With the included Stretch Leveller you can easily adjust the lifting so that the patient lies either horizontally or somewhat higher at the head end.

Lift Sheets

You can choose between a full or a 2-piece lift sheet. The 2-piece lift sheet can be especially suitable in connection with, for instance, gynaecological operations as the lower part can be taken away if necessary.



The lift sheet is attached to the stretcher by inserting the straps in the self-locking feature, which is closed with a light pressure. The lift sheet can be either placed on the bed before the patient lies down or be placed under a patient already lying down.



The fact that LikoStretch 1900 weighs only 19 kg (42 lbs.) makes it easy to handle for one person. It can also be folded for convenient storage.

PRODUCT FACTS

Max. load:	250 kg (550 lbs.)	
Length:	190 cm (74.80")	[folded 69.5 cm (27.36")]
Width:	90.9 cm (35.79")	[folded 90.9 cm (35.79")]
Height:	59.3 cm (23.35")	[folded 69.7 cm (27.44")]
Weight:	19 kg (42 lbs.)	

Product Overview

- 3156051 LikoStretch 1900 incl. Stretch Leveller
- 3685705 LiftSheet 1900, full
- 3685704 LiftSheet 1900, 2-piece



LikoStretch 1900 complies with the requirements for Medical Device Directive Class I products (MDD 93/42/EEC).

Design and Quality by Liko in Sweden.

Liko is quality-certified in accordance with ISO 9001 and its equivalent for the medical device industry, ISO 13485.

Liko is also certified in accordance with environmental standard ISO 14001.



www.liko.com

Manufactured by:

Liko AB
Nedre vägen 100
975 92 Luleå
Sweden
info@liko.se

Local representative in UK

Hill-Rom Limited
Clinitron House, Ashby Park
Ashby de la Zouch,
Leicester LE65 1JG
Tel: 01530 411000
Fax: 01530 411555
info@liko.co.uk

To find your local Liko representative,

please visit www.liko.com or call
+46 (0)920 47 47 00



A **Hill-Rom** Company